FR / DERMAPLIQ® Spray

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Carboxymethylglucose sulfate polymers (RGTA®) are biopolymers, closely related to heparan sulfates in structure. Heparan sulfates provide an anchor point in the extracellular matrix for the physiological factors needed for tissue repair, and participate in the protection of the extracellular matrix needed for the multiplication of cells and restoration of the tissue.

In wounded tissues, heparan sulfates are destroyed by enzymes linked to inflammation. RGTA® replaces the destroyed heparan sulphates, promoting tissue healing. Improving the quality of the extracellular matrix helps reduce discomfort for the animal.

Species:

DERMAPLIQ® spray is suitable for use on cats; dogs; sport and race horses; and exotic pets.

Use:

Depending on the case, it may be very useful to debride the wound, to allow access to living tissue before applying DERMAPLIQ® Spray.

Spray the product onto the wound every 2 days, this interval may be adjusted by your veterinarian.

The duration of treatment can vary from one to several weeks. A 7.5 mL bottle allows about 50 sprays.

Your veterinarian may adapt the instructions for use. Follow their recommendations.

For external use only.

- Clean the wound.
- Spray DERMAPLIQ® onto the wound from a distance of 5 cm.
- Leave for about 1 minute before applying any other product, or applying a dressing.

Storage conditions:

Store the product at a temperature between 2°C and 25°C in its original packaging.

DERMAPLIQ® Spray does not contain colourants or preservatives.

The 3K® spray pump system prevents contamination of the solution by microbes or impurities. Use the product within 6 months of the first spray. Be sure to note the date of the first spray on the package.

Class:

Animal hygiene product, used as an extracellular matrix substitute.

Pack size:

Box of 1 spray bottle of 7.5 ml of sterile solution.

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